



Access to the coaching function

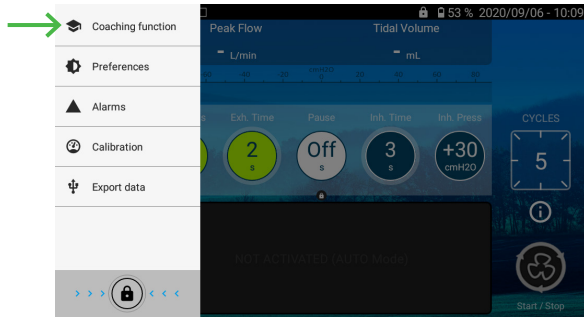

Treatment choice

Select the preset defined by your doctor or physiotherapist

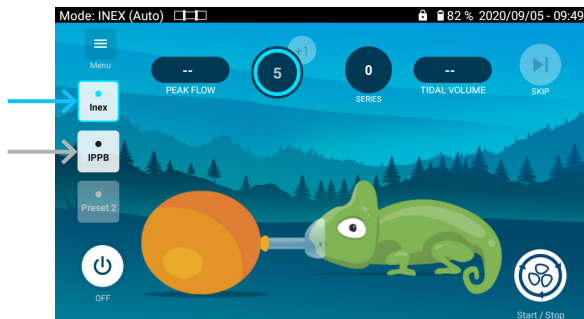

Contact

EOVE
4 Boulevard Lucien Favre
64000 Pau - France

EOVE Phone: +33 (0)5 59 21 86 64

www.device.airliquidehealthcare.com



Air Liquide Healthcare is a world leader in medical gases, home healthcare, hygiene products and healthcare specialty ingredients. It aims to provide customers in the continuum of care from hospital to home with medical products, specialty ingredients and services that contribute to protecting vulnerable lives.



Quick Guide for Patient

Coaching function


Patient's name:

Treatment:

INEX

IPPB


10/2020 - Photos : © Air Liquide - Design CHANH THI

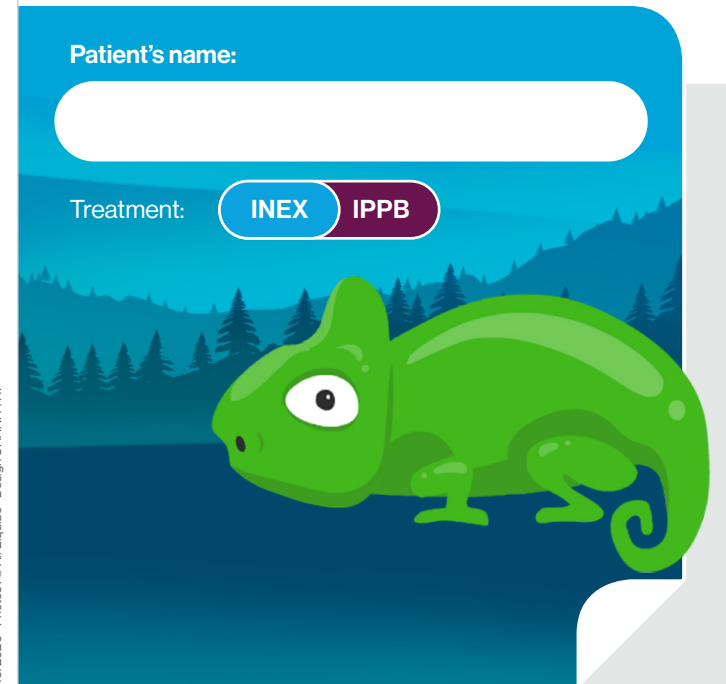

INEX Mode

Yes No

Objective: help you to synchronize yourself with the treatment

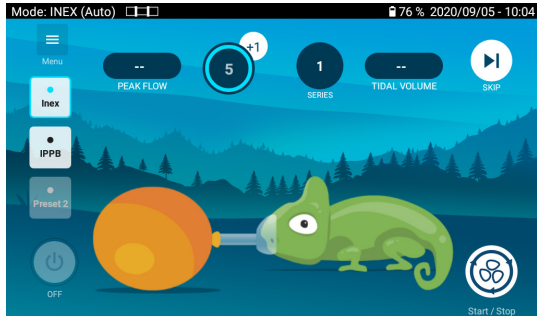


• Start / Stop the treatment

: number of cycles that you have to do

• Possibility to add until 10 cycles anytime during the session

: number of series already done

If needed, at the end of each series, you can start immediately a new one by clicking on "Skip the pause" button

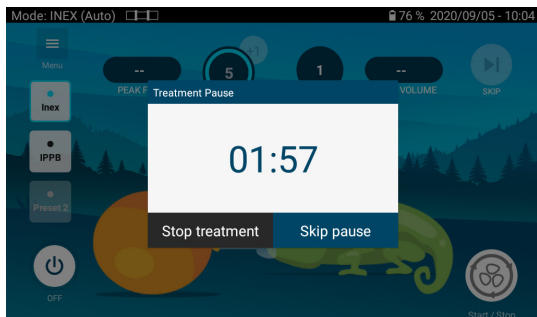

IPPB Mode


Yes No

Objective: permit to your lungs to inhale more air

Use a leak circuit

Beginning of the session:

- 1 Push on Start
- 2 Inhale/Exhale 3 times on your own to determine the target volume of the sessions

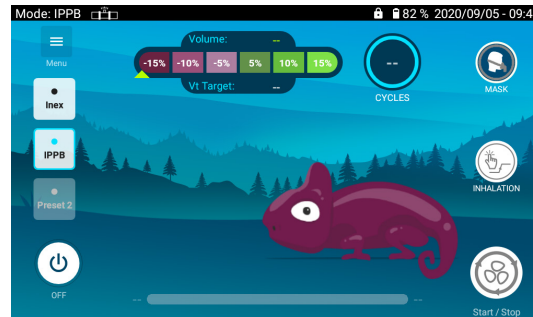

: number of cycles already done

• Leak indicator:
 : green: the position of the mask is good
 : orange: please position the mask better to avoid any leak


: manual trigger of the cycle

• Session time

IPPB Mode

Volume that you inhale during the last cycle

Evolution of the inhaled volume during the cycle

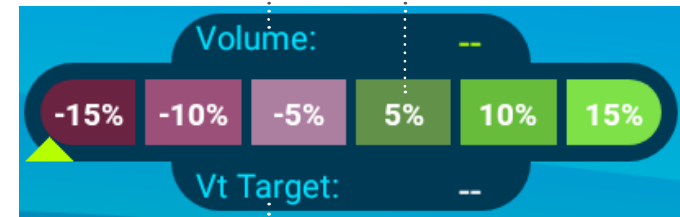

Target volume